

# BD Veritor™ System for Rapid Detection of SARS-CoV-2

## Proper Nasal Swab Sample Collection

REF 256089

1

The BD Veritor System SARS-CoV-2 Kit includes swabs for nasal specimen collection.

2



Carefully insert the swab into one nostril. The swab tip should be inserted up to 2.5 cm (1 inch) from the edge of the nostril. Roll the swab 5 times along the mucosa inside the nostrils to ensure that both mucus and cells are collected.

3



Using the same swab, repeat this process for the other nostril to ensure that an adequate sample is collected from both nasal cavities.

4



Withdraw the swab from the nasal cavity. The sample is now ready for processing using the BD Veritor System SARS-CoV-2 kit.

### Do's and Don'ts of Sample Collection

- Do collect sample as soon as possible after onset of symptoms.
- Do test sample immediately.
- Use only swabs provided with the kit.
- Refer to: Laboratory support for COVID-19 in the EU/EEA at <https://www.ecdc.europa.eu/en/novel-coronavirus/laboratory-support>

For further technical information please contact your local BD representative or visit [bd.com](http://bd.com). For more detailed guidance on the use of the product please refer to the instructions for use.



 Becton, Dickinson and Company  
7 Loveton Circle  
Sparks, Maryland 21152 USA

 Benex Limited  
Pottery Road, Dun Laoghaire  
Co. Dublin, Ireland

BD, the BD Logo, and Veritor are trademarks of Becton, Dickinson and Company or its affiliates. © 2020 BD. All rights reserved.

 [bd.com/e-labeling](http://bd.com/e-labeling)

L012402(01)  
2020-09