


There's a life behind every test

Simplify Flu A+B testing

Streamline your point-of-care flu testing with the portable **BD Veritor™ Plus System**, which features intuitive operation and timely, reliable results.

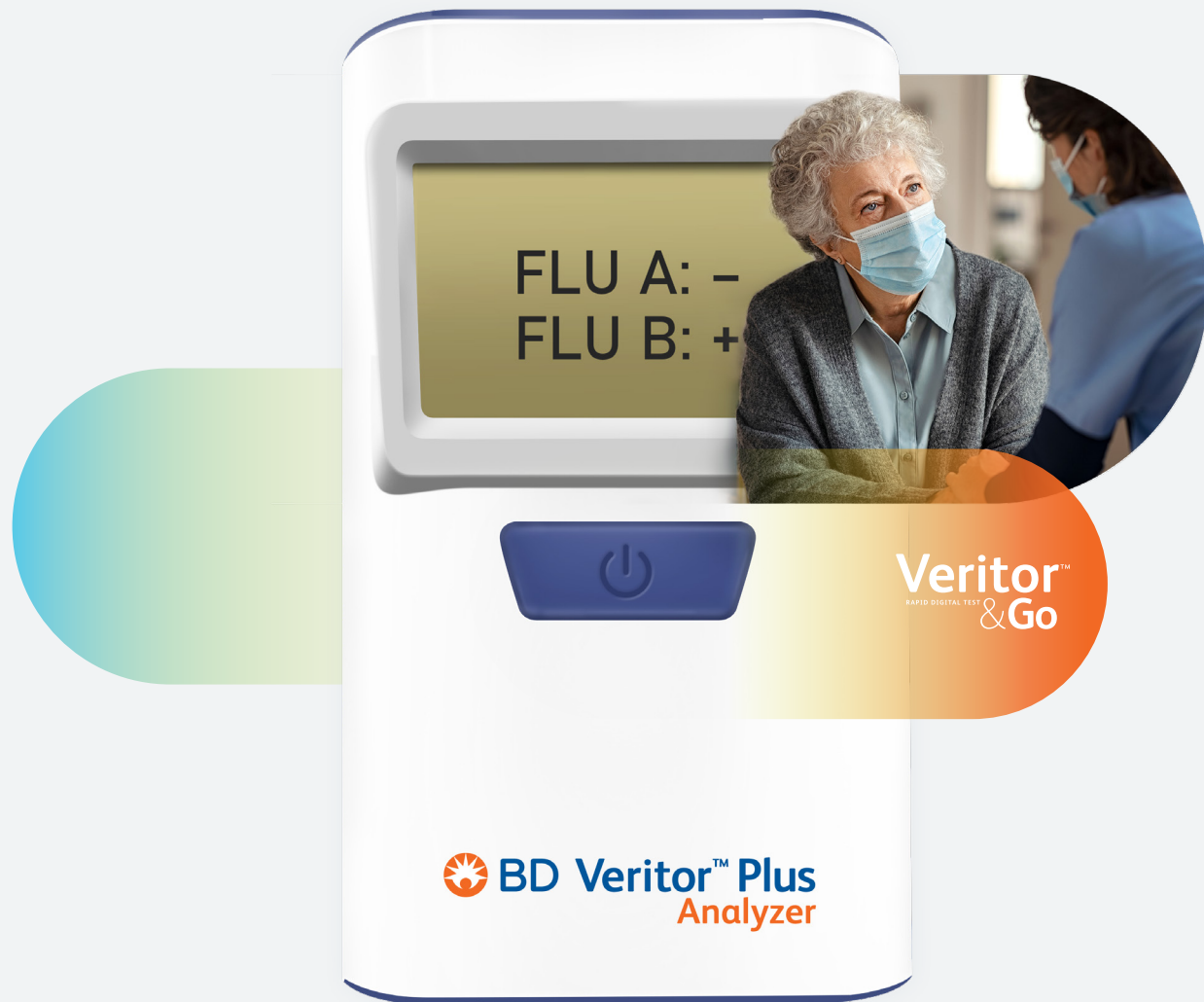

Overcome the challenges of flu season

FLU A+B

The **BD Veritor™ Plus System** offers simple operation, an efficient workflow, and fast, traceable results, which may help improve the patient experience.



Simplifies the testing process

- Helps user operation with single-button functionality that can potentially reduce manual test processing errors
- Provides intuitive sample processing with prefilled, unitized tubes



Delivers workflow efficiency

- Adapts easily to your workflow by offering 2 operational modes
 - » **Walk Away:** the test device is inserted into the Analyzer immediately after preparation, allowing staff to multitask while the sample incubates
 - » **Analyze Now:** the test device is inserted after the incubation time is complete, allowing batches of samples to be tested at once



Achieves reliable, rapid results

- Displays easy-to-read digital results for flu in just 10 minutes¹
- Provides clear, objective results by correcting for nonspecific binding and detecting positives not recognized by the unaided eye¹



Provides result traceability

- Stores or downloads the lot number, patient/specimen ID, operator ID and test records using the BD Veritor™ InfoWiFi Module or via the companion ImageMover application
- Offers result printing capabilities via USB port

Reference: 1. BD Veritor™ System for Rapid Detection of Flu A+B. Package insert. 8087667 Becton, Dickinson and Company.

BD Life Sciences, 2100 Derry Road West, Mississauga, ON L5N 0B3 CA
866-979-9408

Proven performance vs PCR¹

	PPA [†]	NPA [‡]	Results in
Influenza A	83.6% (76.1%, 89.1%)	97.5% (95.7%, 98.5%)	
Influenza B	81.3% (71.1%, 88.5%)	98.2% (95.7%, 99.3%)	

[†]positive percent agreement
[‡]negative percent agreement

Flu A+B results in 10 minutes

Empower your staff with the BD Veritor™ Plus System eLearning online training platform

- Assigns customized training plans by role, product or learning type
- Simplifies learning through 5-minute modules
- Tracks training compliance and competency assessments

Ordering information	Cat no.	Qty
BD Veritor™ Plus System Flu A+B	256045	30 tests
BD Veritor™ Plus System Flu A+B laboratory kit	256041	30 tests
BD Veritor™ Plus Analyzer	256066	1 unit

*BD Veritor™ System for Rapid Detection of SARS-CoV-2:

- This test has been authorized for sale in Canada by Health Canada under Interim Order;
- This test is only authorized for the duration of the Interim Order Respecting the Importation and Sale of Medical Devices for Use in Relation to COVID-19, unless the authorization is terminated or revoked sooner;
- This test has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens.

BDVeritor.com

BD, the BD Logo and Veritor are trademarks of Becton, Dickinson and Company or its affiliates. © 2021 BD. All rights reserved.

