

There's a life behind every test

The BD Veritor™ Plus System simplifies COVID-19* testing at the point of care

Discover the portable, easy-to-use BD Veritor™ Plus System. It provides reliable results in just 15 minutes.



Simplifying COVID-19 testing

COVID-19 (SARS-CoV-2)

When your patients need reliable, rapid COVID-19 testing, turn to the **BD Veritor™ Plus System**. It offers dependable results at the point of care in a simple-to-operate portable instrument.



Simplifies the testing process

- Helps reduce manual test processing errors with its easy operation and single-button functionality
- Provides intuitive sample processing with prefilled, unitized tubes



Achieves reliable, rapid results

- Displays easy-to-read digital results for COVID-19 in 15 minutes¹:
 - » “CoV2: +” for positive; “CoV2: -” for negative
- Provides clear, objective results by correcting for nonspecific binding and detecting positives not recognized by the unaided eye¹



Delivers workflow efficiency

- Adapts easily to your workflow by offering 2 operational modes
 - » **Walk Away:** the test device is inserted immediately into the Analyzer so staff can multitask while the sample incubates (15 minutes)
 - » **Analyze Now:** the test device is inserted after the incubation time is complete, allowing batches of samples to be tested at once (results in seconds)



Provides result traceability

- Stores or downloads the lot number, patient/specimen ID, operator ID and test records using the BD Veritor™ InfoWiFi Module or via the companion ImageMover application
- Offers result printing capabilities via USB port


Reference: 1. BD Veritor™ System for Rapid Detection of SARS-CoV-2. Package insert. 500048916 Becton, Dickinson and Company.

BD Life Sciences, 2100 Derry Road West, Mississauga, ON L5N 0B3 CA
866-979-9408

[BDVeritor.com](https://www.bdveritor.com)

BD, the BD Logo and Veritor are trademarks of Becton, Dickinson and Company or its affiliates. © 2021 BD. All rights reserved.

Clinical performance vs RT-PCR¹

	PPA [†]	NPA [‡]	Results in
COVID-19 (SARS-CoV-2)	91.1% (95% CI, 80.7%-96.7%)	99.6% (95% CI, 97.9%-99.9%)	 15
<small>†positive percent agreement ‡negative percent agreement</small>		COVID-19 results in 15 minutes	

Empower your staff with the BD Veritor™ Plus System eLearning platform

- Assign customized training plans by role, product or learning type
- Simplify learning with 5-minute modules
- Easily track training compliance and competency assessments

Ordering information	Cat no.	Qty
BD Veritor™ Plus System SARS-CoV-2 Kit	256089	30 tests
BD Veritor™ Plus Analyzer	256066	1 unit

¹BD Veritor™ System for Rapid Detection of SARS-CoV-2:

- This test has been authorized for sale in Canada by Health Canada under Interim Order;
- This test is only authorized for the duration of the Interim Order Respecting the Importation and Sale of Medical Devices for Use in Relation to COVID-19, unless the authorization is terminated or revoked sooner;
- This test has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens.

